1. Indication
printo[®] GR-18.2 IB is a resin for the generative production of dental indirect bonding trays based on image projection systems.

2. Contraindication

printo[®] GR-18.2 IB is contraindicated ...
1. ... if a patient is known to be allergic to any of the ingredients.
2. ... direct intraoral contact of uncured or partially uncured material.
3. ... for every application that is not part of the indication (see above).

3. Patient Target Group

Persons being treated in the context of a dental procedure.

4. Intended Users

Dentist, dental technician

5. Requirements

Software – information available from:

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Hardware (3D Printing) – information available from:

ASIGA - 2/19-21 Bourke Road, Alexandria, NSW 2015, Australia

Hardware (Post Curing) – information available from:

pro3dure medical (CD-1, CD-2) - Am Burgberg 13, D-58642 Iserlohn, Germany

Additional specifications on www.pro3dure.com.

6. Material

printo[®] GR-18.2 IB consists of functional (meth)acrylic resins.

7. Geometric presetting

According to the software design specifications.

8. Material parameters

Radiation penetration depth controllable by exposure time

50 µm

100 µm

9. Manufacturing process (fig. 1-10)

1. Prepare data (CAD & build preparation).
2. Choose process parameters (build style, etc.).
3. Transfer prepared data to 3D printing device.
4. Prepare 3D printing – shake bottle.
5. Fill resin tank of 3D printing device.
6. Build the parts.
7. Clean parts (with IPA ≥ 97 % or equivalent cleaning agent) approx. 4 min. in an ultrasonic bath or equivalent device – precleaning recommended.
8. Dry parts (with compressed air until there are no residues of IPA or equivalent cleaning agent).
9. Post curing (10 min.): inert atmosphere recommended (use adequate light curing device).
10. Finish parts.

10. Finishing processes

Not applicable

11. Notice

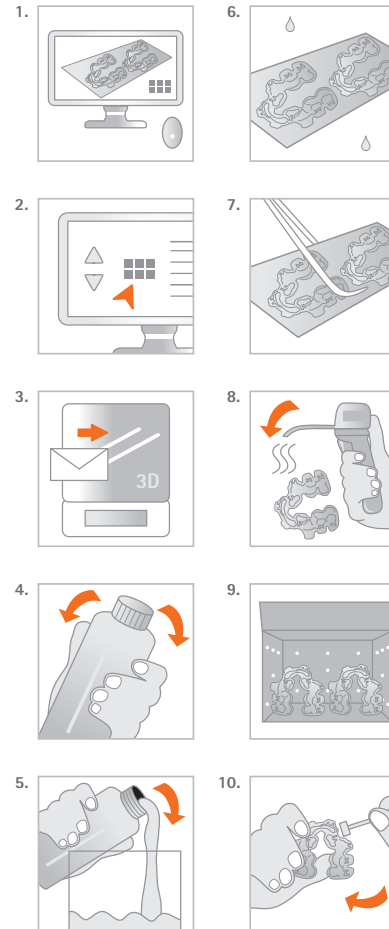
Please follow the instructions provided by the software manufacturer in terms of parameter settings and design recommendations. Please follow the instructions provided by the hardware manufacturer in terms of parameter settings/printing & post curing recommendations. To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances. Deviations from the described manufacturing processes or storing conditions may lead to different mechanical and optical properties of the material. Ensure personal protective gear during processing. According to the EU Medical Devices Regulation, users/patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred. **Caution:** The lot number and the best before date are indicated on each packaging. In case of claims please always indicate the lot number of the product. Do not use the product after expiry of the best before date. Avoid breathing dust/fume/gas/mist/vapours/spray. Use only outdoors or in a well-ventilated area. Contaminated work clothing should not be allowed out of the workplace. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. IF INHALED: Remove person to fresh air and keep comfortable for breathing. Call a POISON CENTER or doctor/physician if you feel unwell. Specific treatment (see information on this label). If skin irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash it before reuse. Collect spillage. Dispose of contents/container according to official regulations.

12. Hazard statements

May cause an allergic skin reaction. Harmful if inhaled. Toxic to aquatic life with long lasting effects.

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Distributor (US):
pro3dure medical LLC
9825 Valley View Road · Eden Prairie, MN 55344
Phone: 952-426-1928 · Fax: 952-681-7515
info@pro3dure.com · www.pro3dure.com · Made in Germany

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